

WARNING: TO PREVENT FIRE OR SHOCK HAZARD, DO NOT EXPOSE THIS APPLIANCE TO RAIN OR MOISTURE.

SONIC ALERT'S 1 YEAR LIMITED WARRANTY

The Sonic Boom Alarm Clock Model SBH400ss is warranted against manufacturing defects in materials and workmanship for (1) year from the date of purchase. Within this period Sonic Alert will repair or replace at our option the SBH400ss without charge for parts and labor. Simply send the SBH400ss (postpaid) and a copy of your sales slip as proof of purchase to:

Sonic Alert Inc., 1050 E. Maple Road, Troy MI 48083

This device complies with part 15 of the Federal Communications Commission Rules. Operation is subject to the condition that this device does not cause harmful interference.

Note: The manufacturer is not responsible for any radio or TV interference caused by unauthorized modifications to this equipment. Such modification could void the user's authority to operate the equipment.



Made in china

Sonic Alert 1050 East Maple Rd. Troy, MI 48083

www.SonicAlert.com 1-800-566-3210 V/TTY

093-HK205-003



Sonic Alert

America's Leader in Visual Alerting Systems.

**Sonic Boom Alarm Clock & Bed Shaker
Model SBH400SS**

Photo

OPERATION INSTRUCTIONS

IMPORTANT- Please read these instructions carefully before use and retain for future reference. See operating instructions on back.

Warning: To prevent fire or shock hazard do not expose this appliance to rain or moisture. Before connecting to mains make sure that the requirements specified on the apparatus corresponds to the available power supply i.e. AC 120V/220-240V (50Hz/60Hz)

GENERAL DATA

Power: Consumption: 8w, 120v / 60 Hz (Adaptor AC ~9V 750 MA)
UL listed transformer (Underwriters Laboratories)

Made in china


Sonic Alert 1050 East Maple Rd, Troy, MI 48083

www.SonicAlert.com 1-800-566-3210 V/TTY

INSTRUCTIONS

LOUD -VIBRATING ALARM CLOCK WITH BED SHAKER

MODEL: SBH400SS

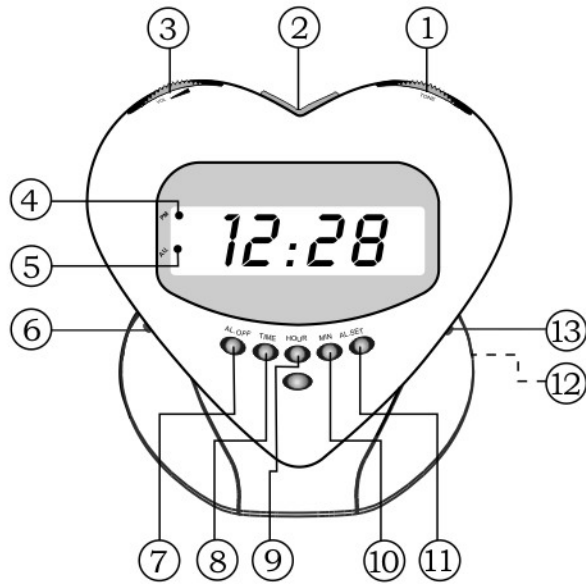

- 1) Volume Control
- 2) Snooze Button
- 3) Tone Control
- 4) PM Indicator
- 5) Alarm Indicator
- 6) Dimmer Switch
- 7) Alarm Off Button
- 8) Time Set Button
- 9) Hour Button
- 10) Minute Button
- 11) Alarm Set Button
- 12) Backup Battery Compartment
- 13) Function Switch
- 14) AC Power Input
- 15) Vibrator Input

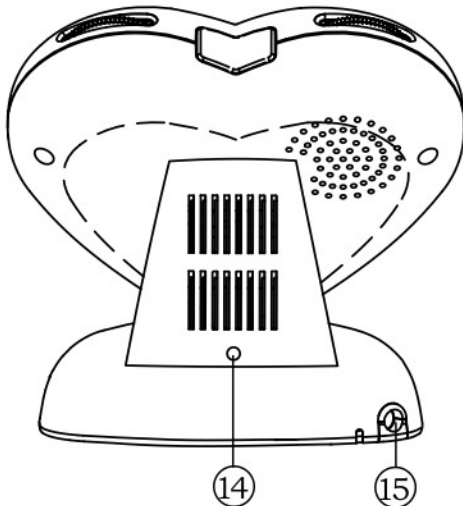


Figure B

Setting the correct time-

- 1) Press and hold the "TIME" set (8) button.
- 2) Depress hour button (9) as necessary to advance the clock display until the hour is correct.
- 3) Depress minute button (10) as necessary to advance the clock display until the minute is correct.
- 4) Once the time is set, release the "TIME" set (8) button.
- 5) Lighted dot (4) in upper left clock display indicates time is PM, a dot does not appear for AM.

Setting the alarm time-

- 1) Simply follow the instructions mentioned under " Setting the correct time" except that you will have to Press and hold the "ALARM" set (11) button.
- 2) Make sure the alarm time is correctly set for AM or PM lighted dot (4) indicates PM. Be sure to release the HOUR and MIN set buttons before releasing the ALARM button.
- 3) Once the time is set, release the "ALARM" set (11) button.
- 4) To enable the alarm function, slide the Function Switch (13) to the desired position. This will turn on the alarm feature. Otherwise, alarm will not go off for the alarm time set.
- 5) You will see a dot (5) on the left side of the LED display to show the alarm has been enabled.

Alarm-

Slide the "FUNCTION SWITCH" (13) up to the position that you desire to wake to, you can choose "VIB" for bed shaker, "BUZZ" for sound, and "VIB/BUZZ" for both. NOTE: whichever function you choose to wake to, will be the same function for both alarms.

Snooze Operation-

SNOOZE button (2) - press after alarm has sounded. The alarm will be turned off for 9 minutes for extra sleep. The alarm will come back again after 9 minutes. You may repeat up to 119 minutes.

Shutting Off Alarm-

Alarm can be shut off temporary for the day or permanently shut off.

- 1) When the alarm goes off, simply press the AL off button (7) the alarm will shut itself off for the day and come back on the next day at the same time if alarm setting has not changed
- 2) To shut the alarm off permanently, slide function switch (13) to off position. Alarm will not come on until Function Switch (13) is set to "BUZZ" or "VIB" or "VIB/BUZZ" position.

Volume Control-

To adjust the volume rotate turn the wheel located on top right side (1) of the clock.

Tone Control-

To adjust the "TONE CONTROL" (3) turn the wheel located on top left side (3) of the clock to the desired tone sound.

Bed Shaker / Vibrator- Figure B

- 1) Attach the bed shaker jack to the back of clock at VIBRATOR input (15).
- 2) To turn on vibrator slide FUNCTION switch (13) to VIB or VIB/BUZZ position.
- 3) Place vibrator under mattress or pillow.

Battery Backup-

To use battery backup, you can install a 9-volt alkaline battery in the back of the clock located under the battery cover (12). Battery not included. Note the LED display will not show during power failure but the clock will still be running and keeping proper time until the power comes back.

AC-9V Power Input -

Attach the AC power supply input to the back of clock at AC-9V IN (14) next to the VIBRATOR input, smaller input of the two.

NOTE: IMPORTANT: do not confuse "AC POWER INPUT"(14) with the "VIBRATOR INPUT" (15).